



## METAL PRIMER

Sci-Pharm Catalog No. 70-10

### KIT CONTAINS:

- Primer, 15cc

### INDICATIONS FOR USE:

A metal primer for in situ repair of damaged or worn veneers on cast restorations.

### CONTRAINDICATIONS:

There are no contraindications.

**CAUTION: Contains flammable solvent.**

### INSTRUCTIONS FOR USE

Using a fine brush, apply a thin layer of *CuRAY-PriMet*® on washed, roughened, and blow-dried exposed metal surface. Wait one minute. Cure for ten seconds with visible light curing instrument. Follow with the application of light-cured restorative.

### HELPFUL HINTS

The use of paint-on (flowable) light-cured restoratives, such as Scientific Pharmaceuticals' *CuRAY-Match*®, is strongly preferred over conventional filling restoratives.

Extend the restoration over a relatively large area of porcelain ground down to feather edge around the area to be restored.

### STORAGE AND SHELF-LIFE

Store at temperatures not exceeding 75°F (24°C). When stored under such conditions, the material has a shelf-life of two years.



Qarad EC - REP BV  
Pas 257  
2440 Geel  
Belgium

For technical information, call or write:

Quality Management System Certified to

**ISO 13485:2016**



SCI-PHARM®



SCIENTIFIC PHARMACEUTICALS, INC.

3221 PRODUCER WAY • POMONA, CALIFORNIA USA 91768

PHONE: (800) 634-3047 • (909) 595-9922

FAX: (909) 595-0331 • E-MAIL: [scipharm@msn.com](mailto:scipharm@msn.com)

WEBSITE: <http://www.scipharm.com>

6079 - P1/2

Printed in U.S.A.

100121

...chemistry working for medicine and dentistry...

*Trust these trademarks...*

 **CuraRAY<sup>®</sup>-FIL<sup>®</sup>**

**GLASION<sup>®</sup>**

**LUSTER<sup>®</sup>**

 **CuraRAY<sup>®</sup>-MATCH<sup>®</sup>**

**TEMPO-REX<sup>®</sup> MICROJOIN<sup>®</sup>**

**CEMPER<sup>®</sup> DIAMOND LUSTER<sup>®</sup>**

 **CuraRAY<sup>®</sup>-SUPPORT<sup>®</sup>**

**FloroPhos<sup>®</sup>**

**PCURE-ON-TOUCH<sup>®</sup> L<sup>®</sup> UNTIL<sup>®</sup>**

**NEWCORE<sup>™</sup>**

**FLUOROSEAL<sup>®</sup>**

 **CuraRAY<sup>®</sup> - PriMet<sup>®</sup>**

**FASTFIL<sup>®</sup>**

**SCI-SPAN<sup>®</sup>**

**LC SUPPORT<sup>®</sup>**

 **CuraRAY<sup>®</sup> - II<sup>®</sup>**

**SUPPORT<sup>®</sup>**