

StikFree[™]

Releasing Agent

Cat. No. 70-05/71-05

CONTENTS: Cat. No. 70-05 - 15cc *StikFree*[™]
Cat. No. 71-05 - 6cc *StikFree*[™]

StikFree[™] is an original formulation developed specifically to facilitate application of dental restoratives or cements which tend to stick to plastic and metal dental instruments. When used as directed, *StikFree*[™] will not alter or otherwise impair the properties or performance of these materials.

DIRECTIONS FOR USE

Dispense one drop of *StikFree*[™] on a mixing pad. Wet all instrument surfaces that are likely to come into contact with the material. Remove any excess by gently scraping against the edge of the pad.

STORAGE AND SHELF-LIFE

Store at temperatures not exceeding 75°F (24°C). When stored under such conditions, the material has a shelf-life of two years.

For technical information or to order, contact:
Scientific Pharmaceuticals, Inc.

3221 Producer Way • Pomona, California, U.S.A. 91768
Phone: (909) 595-9922 • (800) 634-3047 • Fax: (909) 595-0331
E-Mail: scipharm@msn.com • Website: www.scipharm.com

Trust these trademarks...

 **CURAY-FIL®**

GLASION®

LUSTER®

 **CURAY-MATCH®**

TEMPO-REX® MICROJOIN®

CEMPER® DIAMOND LUSTER®

 **CURAY-SUPPORT®**

FloroPhos®

PCURE-ON-TOUCH L® UNTIL®

NEWCORE™

FLUOROSEAL®

 **CURAY - PriMet®**

FASTFIL®

SCI-SPAN®

LC SUPPORT®

 **CURAY - II®**

SUPPORT®