



Manufactured in U.S.A. by:

SCI-PHARM



Sci-White™

Teeth Whitener
Sci-Pharm Catalog No. 85-20

TECHNICAL BULLETIN
Instructions

Sci-White™
Teeth Whitener

Cat. No. 85-20

**FOR APPLICATION BY A PROFESSIONAL OR
UNDER PROFESSIONAL SUPERVISION.**

KIT CONTAINS:

- Whitener, 12g in Four Syringes
- 12 Dispensing Tips
- 2 Tray Forming Sheets
- Instructions



**OUTSTANDING FEATURES
OF THE MATERIAL**

- Accelerated action = Shortened application time
- Low irritation potential = Reduced patient discomfort
- Pleasant mint flavor = Enhanced patient acceptance
- Amine free = Absence of questionable ingredients

INDICATIONS FOR USE:

Lightening of vital and nonvital teeth.

CONTRAINDICATIONS: This product is not recommended for:

- Pregnant or lactating women.
- Heavy smokers or persons having habits of high alcohol consumption.
- Children below age 15 (although no significant adverse effects have been reported, little information has been published related to application of teeth whiteners on children).
- Users having inflamed or scarified oral tissues, leaking restorations or decayed teeth.

Warning: Occasionally, patients may experience temporary discomfort in the teeth and gingiva. If discomfort persists, discontinue treatment.

CAUTION

Federal law restricts this device to sale
by or on the order of a dentist.

INSTRUCTIONS FOR USE

It is recommended that patients receive a prophylaxis and a dental examination prior to beginning the whitening treatment. A properly fabricated tray will improve the effectiveness of the whitening process and contribute to patient comfort. It is also important to explain proper application techniques to the patients to enhance the efficacy and comfort during home treatments.

Treatment Procedures:

1. Prior to first treatment, procure the trays and adjust to achieve a good fit.
2. Apply a small amount of tooth whitening material to the facial side of each tooth space within the tray.
3. Seat custom trays into place over the teeth. Do not squeeze the trays.
4. Remove any excess material from gingiva using a soft toothbrush, tissue, or finger.
5. The trays should be worn for 30-150 minutes, unless the dentist has instructed otherwise.
6. Remove the trays.
7. Brush teeth using a soft toothbrush and rinse mouth thoroughly.
8. Clean trays using a soft toothbrush and water.
9. Store trays in the tray case.
10. Repeat the procedure daily or as instructed by the dentist.

INGREDIENTS

Carbamide Peroxide (16%) - An Active Ingredient, Glycerin, Buffered Polyacrylic Acid, Peppermint Oil.

ADDITIONAL NOTES

1. Do not eat, drink, or smoke while wearing the trays.
2. Keep out of reach of children.
3. Wash spills immediately.

STORAGE AND SHELF-LIFE

The shelf-life of the material is six months when stored at temperatures not exceeding 73° F (23° C).
Constant storage under refrigeration will extend the shelf-life of the material to one year.

© Scientific Pharmaceuticals, Inc., 2006



Qarad EC - REP BV
Pas 257
2440 Geel
Belgium

For technical information, call or write:

Quality Management System Certified to

ISO 13485:2016



SCIENTIFIC PHARMACEUTICALS, INC.
3221 PRODUCER WAY • POMONA, CALIFORNIA USA 91768
PHONE: (800) 634-3047 • (909) 595-9922
FAX: (909) 595-0331 • E-MAIL: scipharm@msn.com
WEBSITE: <http://www.scipharm.com>