

PLASTIC BRACKET PRIMER

For Priming Plastic Brackets in Conjunction with
Light-Cured Orthodontic Adhesives

Sci-Pharm Catalog No. 60-055 B

INDICATIONS FOR USE:

For use in promoting adhesion to plastic orthodontic brackets in direct bonding applications with self-cured adhesives (such as Scientific Pharmaceuticals' *CureRAY-Eclipse*®, Cat. No. 60-05).

CONTRAINDICATIONS:

This product should not be used where patients have known hypersensitivity to methacrylate monomers.



INSTRUCTIONS FOR USE

Cat. No. 60-055 B

NOTE: Shake vigorously before each use.

Using a disposable brush, apply a thin layer of plastic bracket primer to the bases of the brackets. Wait 90 seconds before proceeding with bonding. Avoid exposure of brackets to bright light before bonding.

CAUTION: Flammable liquid. Avoid contact of the primer with skin. Repeated or prolonged exposure to the primer may cause irritation of more sensitive skin. If contact occurs, wash immediately with cold water and soap.

STORAGE AND SHELF-LIFE

Store at temperatures not exceeding 75°F (24°C), ensuring that the cap is securely in place. When stored under such conditions, the material has a shelf-life of one year.

OUTSTANDING SCI-PHARM BRACKET BONDING SYSTEMS

CuRAY-Eclipse®: This state of the art light-cured orthodontic adhesive works well with conventional curing lights as well as laser curing instruments.

Cure-On-Touch®: This fast, easy-to-use, economical no-mix system provides a strong, reliable bond.

Federal law restricts this device to sale by or on the order of a dentist.



For technical information, call or write:

Quality Management System Certified to

ISO 13485:2016



SCIENTIFIC PHARMACEUTICALS, INC.
3221 PRODUCER WAY • POMONA, CALIFORNIA USA 91768
PHONE: (800) 634-3047 • (909) 595-9922
FAX: (909) 595-0331 • E-MAIL: scipharm@msn.com
WEBSITE: <http://www.scipharm.com>

6733 - P1/2

Printed in U.S.A.

092624

...chemistry working for medicine and dentistry...

Trust these trademarks...



GLASION®

LUSTER®



TEMPO-REX® MICROJOIN®

CEMPER® DIAMOND LUSTER®



FloroPhos®

PCURE-ON-TOUCH L® UNTIL®

NEWCORE™

FLUOROSEAL®



FASTFIL®

SCI-SPAN® LC SUPPORT®



SUPPORT®