



Manufactured in U.S.A. by:

**SCI-PHARM**



Cat. No. 62-011 B  
**BAND SECURE™**  
Light-Cured Orthodontic  
Band Cement

**TECHNICAL BULLETIN**  
Instructions

**BAND SECURE™**

Orthodontic Band Cement with Fluoride

**KIT CONTAINS:**

- 5g Cement in Syringe-type Dispenser
- Instructions



Cat. No. 62-011 B

**OUTSTANDING FEATURES  
OF THE MATERIAL**

- Virtually unlimited working time.
- Ease of handling.
- Economy of use – minimum waste.
- Highly reliable.
- Time savings, fast application.
- Contains leachable fluoride.
- Long shelf life with no need for refrigeration.

© Copyright 2007 Scientific Pharmaceuticals

**CAUTION**

**A. TOOTH AND BANDS PREPARATION**

- After removal of separators, scale gingival margins.
- Prophy teeth to be banded. Use prophy pastes that do not contain oils or petrolatums. Rinse and dry the teeth thoroughly.
- Trial fit bands. Clean bands before cementing. Use of bands with mechanically roughened or etched inside surfaces is recommended.

**B. CEMENTING**

- Isolate teeth to be banded.
- Squeeze the cement slowly from the syringe dispenser. Using a small spatula, or similar tool, cover the entire inside surface of the band with a thin layer of the cement. **While performing this procedure, shield the cement from direct exposure to intense light to avoid premature cure of the cement.** After dispensing, promptly replace the cap on the syringe.
- Make sure the tooth to be banded is dry and firmly seat the band. Remove excess cement with a scaler or cotton roll.
- Cure the cement from the occlusal side for 40 – 50 seconds (depending on light intensity) with conventional dental curing light, or 5 – 10 seconds when using high speed curing instrument.

---

**STORAGE AND SHELF LIFE**

Store at temperatures not exceeding 75°F (24°C). When stored under such conditions, the material has a shelf life of two years.

*For technical information, call or write:*



*Quality Management System Certified to*

**ISO 9001 and ISO 13485**

**CE Marked Products**

**SCIENTIFIC PHARMACEUTICALS, INC.**

3221 PRODUCER WAY • POMONA, CALIFORNIA USA 91768

PHONE: (800) 634-3047 • (909) 595-9922

FAX: (909) 595-0331 • E-MAIL: [scipharm@msn.com](mailto:scipharm@msn.com)

WEBSITE: <http://www.scipharm.com>