

StikFree™ Releasing Agent Sci-Pharm Catalog Nos. 70-05 & 71-05

KITS CONTAIN:

Cat. No. 70-05: StikFree™, 15cc
 Cat. No. 71-05: StikFree™. 6cc

StikFree™ is an original formulation developed specifically to facilitate application of dental restoratives or cements which tend to stick to plastic and metal dental instruments. When used as directed, StikFree™ will not alter or otherwise impair the properties or performance of these materials.

INSTRUCTIONS FOR USE

Dispense one drop of *StikFree*[™] on a mixing pad. Wet all instrument surfaces that are likely to come into contact with the material. Remove any excess by gently scraping against the edge of the pad.

STORAGE AND SHELF-LIFE

Store at temperatures not exceeding 75°F (24°C). When stored under such conditions, the material has a shelf-life of two years.

NOTE: Not a medical device.



For technical information, call or write:



Quality Management System Certified to

ISO 13485:2016

SCIENTIFIC PHARMACEUTICALS, INC.
3221 PRODUCER WAY • POMONA, CALIFORNIA USA 91768
PHONE: (800) 634-3047 • (909) 595-9922

FAX: (909) 595-0331 • E-MAIL: scipharm@msn.com WEBSITE: http://www.scipharm.com

6165 - P1/2 Printed in U.S.A. 062821

Trust these trademarks...

CURAY -FIL GLASION

LUSTER® CURAY -MATCH

TEMPO-REX' MICROJOIN'

CEMPER DIAMOND LUSTER

CURAY SUPPORT FloroPhos

PCURE-ON-TOUCH TOUCH INTIL

NEW CORE[™]

FLUOROSEAL®

CURAY - PriMet® PASTINIT,

SCI-SPAN® LC STPPORT®

CURAY - II SUPPORT

6165 - P2/2 Printed in U.S.A. 062821